





As a Full Service Digital Dental Laboratory, **Milident Inc.** accepts files from all scanners. Find your scanner below, and follow the instructions, in order to start sending us your cases today!

MEDIT Link

- Go to www.meditLink.com
- Log in with your username and password
- Click on Partners  search cadcam@milident.com
- Send a request by clicking 

Once we accept your request you will be able to “order” your case after scanning

iTero

- Call iTero support at 1.800.577.8767
- Select option 1, then option 1
- Request to have our lab added to scanner
- Provide our lab name, **Milident Inc.**, and the iTero customer number 147
- On the scanner - go to settings and select “Sync Configuration”

Carestream

- Visit **Carestream connect** on your scanner
- Search for **Milident Inc.** (cadcam@milident.com)
- Add Lab
- Select **Milident Inc.** when submitting scans or raw .STL files
- Email your favorite digital lab, **Milident Inc.**



- Log in to Cerec Connect at www.sirona-connect.com
- Select “My Cerec Connect” then “Edit Account”
- Under “My Account” select “My Favorite Laboratories”
- Type in name Milident or Milident Lab (cadcam@milident.com) in the field for “Company name” then the Postal Code
- Click “Find”
- Check the box beside Capital Ceramics Ltd.
- Click “Add” to complete the registration

| TRIOS

- Useful link: [3Shape TRIOS - Making connections - YouTube](#)
- Log in to 3Shape software or link: <https://portal.3shapecommunicate.com/login>
- Click “More” and select “Settings”
- Then in the menu click “Connections”
- Click on “Labs” then the “Add”
- Search Milident Inc. by email address: cadcam@milident.com
- Click “Connect” to send a connection request
- We will approve your request ASAP

Once we approved your connection request it will be added to your list of connected Labs

- Click on “Patients” and “Save” your changes

